

The 91443C Flexport® Interface lets information from a Dräger Anesthesia Machine be shown on Spacelabs Healthcare monitors. The Flexport Interface provides bedside and central station display of patient data and is compatible with XPREZZON®, Qube® monitors, and ICS G2. The data given by the interface is available for export to other systems through the ICS HL7 interface.

**Note:**

91443C does not support Ultraview products. If the Flexport is directly attached to an Ultraview monitor, or to a monitor with unsupported software version, the Flexport shows a message, **This monitor version does not support the 91443C Flexport.**

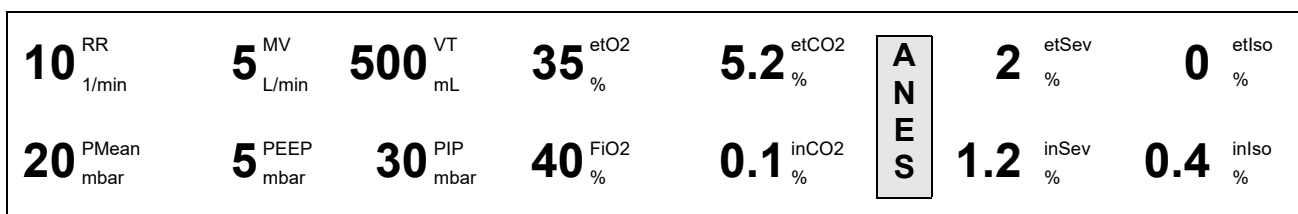Bedside view with no waveform selected. This is the default when the Flexport starts.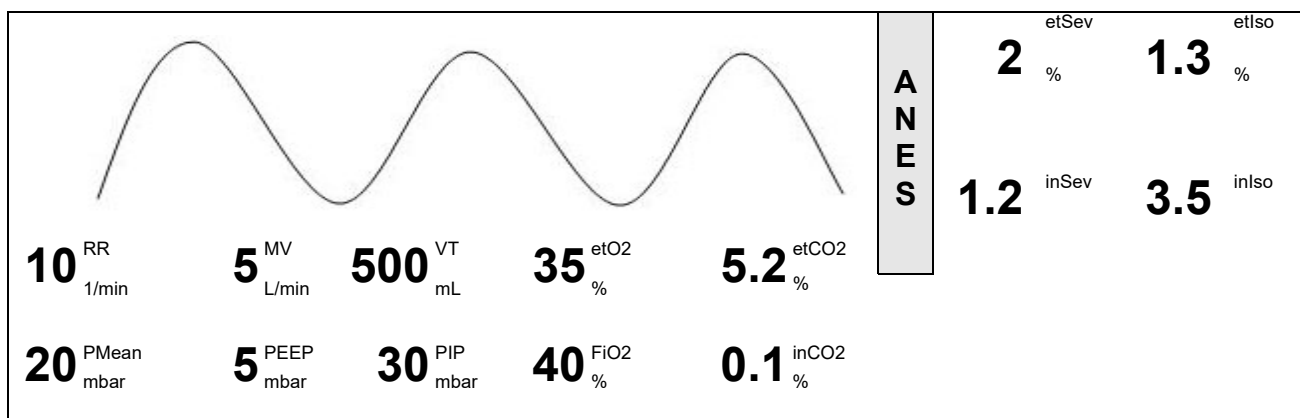

Bedside view with CO<sub>2</sub> Waveform selected with a primary and secondary agent

## Features

**Parameters** Each Flexport interface counts as one parameter when computing parameter capacity for monitors.

## Product Specifications

### Physical Dimensions

Height	2.5 cm (1 in)
Width	13.5 cm (5.3 in)
Depth	4.6 cm (1.8 in)
Weight	0.11 kg (4 oz)

## Environmental Requirements

### Operating

Temperature	10° to 40° C (50° to 104° F)
Humidity	95% (non-condensing)
Altitude	0 to 2,000 m (0 to 6,562 ft)

### Storage

Temperature	-40° to 75° C (5° to 167° F)
Humidity	95% (non-condensing)
Altitude	0 to 12,192 m (0 to 40,000 ft)

## Compatibility

### Monitors Supported

XPREZZON <sup>®</sup> (91393) Qube <sup>®</sup> (91390)	Version 3.05.10 or later  <b>Note:</b> <i>XPREZZON and Qube trends are supported.</i>
ICS G2	Version 4.02.02 and 4.03 or later

### Monitors Not Supported

- XprezzNet
- Note:**  
*Channel handlers created for XprezzNet interfaces are not available.*
- mCare (91220), Maternal Obstetrical Monitor™ (94000)
  - Ultraview SL products:
    - SL2200 (91367)
    - SL2400 (91369)
    - SL2600 (91370)
    - SL2700 (91387-27)
    - SL2800 (91387-28)

## Software and Protocol List

**Note:**

Spacelabs Healthcare confirms that the Flexport software versions shown in the table that follows are verified for use with the software or protocol versions listed for the source devices

Company, Product, or Source Device	Software and Protocol Version	Spacelabs Healthcare Flexport Software Version	Flexport Model Number
Apollo	Version 4.50 operating system Medibus 6.00 protocol	1.00.01	91443C
Primus	Version 4.50 operating system Medibus. 6.00 protocol	1.00.01	91443C
Primus IE	Version 4.50 operating system Medibus 6.00 protocol	1.00.01	91443C
Perseus	Version 1.12 operating system Medibus.X protocol	1.00.01	91443C

## Accessories

### Connectors

Female	6-pin modular jack
Male	9-pin SDLC connector

### Cable Length

SDLC	30.48 cm (12 in)
RS-232	2.44 m (96 in)

### Maximum SDLC Cable Length

The total length of all SDLC cables from the power source (monitor or module housing) to the Flexport Interface should not exceed 1.8 m (6 ft).

When linking other Spacelabs Healthcare devices, the Flexport Interface must be the closest device to the monitor or module housing.

Description	Part Number
Installation Kit for Dräger Flexport	045-0152-00
Modular Cable - Cable Assy, 6c w/shld, RS232 Mod, EMC	012-0182-02
Transition Connector - 9 to Mod	131-1755-01
Transition Connector - 9 to Mod (Dräger Perseus)	131-0686-02
Cable Assembly, SDLC, Flexport - for qube 91390	012-0152-00
Terminator Plug, SDLC, 9pin	012-0507-02
Cable Assembly, SDLC, Flexport - for XPREZZON 91393	012-0555-00

## Documentation

---

CD-ROM Part Number	<i>91443C Flexport® Interface Dräger Apollo/Primus/Primus IE/Perseus CD-ROM</i> (P/N 084-1520-xx)
OEM Software Compatibility	<i>Spacelabs Healthcare Connectivity</i> ( <a href="https://www.spacelabshealthcare.com/products/patient-monitoring-connectivity/networking-and-integration/flexports/">https://www.spacelabshealthcare.com/products/patient-monitoring-connectivity/networking-and-integration/flexports/</a> )
Supplies and Accessories	<i>Spacelabs Healthcare Supplies and Accessories Catalog</i> ( <a href="sa.spacelabshealthcare.com">sa.spacelabshealthcare.com</a> )

---

## Regulatory Approvals

CE marked in accordance with the Medical Device Directive 93/42/EEC.

*Please refer to <http://www.spacelabshealthcare.com/en/company/trademarks> for a full listing of Spacelabs Healthcare trademarks. Other brands and product names used herein are trademarks of their respective owners.*

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